

Strumen[™] 2%
Strumen[™] 2.45 %



**High level Disinfectant and sterilising
solution for Surgical Instruments**



Strumen™ 2% & Strumen™ 2.45 %

Strumen™ is a Standard instrument disinfectant with a standard formula, which remains fully active for upto 28 days after activation. Suitable for treatment of contaminated instruments. Focuses on protecting patients and staff, without affecting the quality of instruments.

Composition

STRUMEN™ 2% & 2.45 % Contains

- ❖ Glutaraldehyde solution - 2 % w/v with Activator
- ❖ Glutaraldehyde solution - 2.45% w/v with Activator

Strumen™ 2 % & Strumen™ 2.45 % is

- ❖ Bactericidal, Fungicidal, Virucidal Pseudomonocidal, (including HIV, HBV and Herpes) action in just 10 minutes.
- ❖ Biocidal action against resistant strains of Mycobacterium TB in 20 minutes.
- ❖ Sporocidal action in 10 hours.

Compatible with Materials

Modern instruments & endoscopes are made particularly of different material combinations. Strumen™ 2% & 2.45% has excellent compatibility with glass, ceramic, stainless steel, nonferrous metal, aluminium, hard rubber & latex, silicon as well as synthetic materials.

Precautions

- ❖ Avoid direct contact with Skin and Eyes, In case of contact with eye, flush with water and seek medical assistance.
- Don't immerse carbon steel objects for more than 24 hrs.
- ❖ Wearing hand gloves and face mask is recommended. Use in a well ventilated area
- ❖ Keep lid on soaking trays at all times except when instruments are removed or added to the solution.

Instructions for use

For Complete sterilization or disinfection of medical instruments and equipments, thoroughly rinse with water and rough dry objects before immersing in STRUMEN™ 2% and STRUMEN™ 2.45% activated solution. Cleanse and rinse the lumens of hollow instruments before filling with activated STRUMEN™ solution



SANMED
ACTIVE HYGIENE

Sanmed Healthcare Pvt Ltd.,

401, Abhis Hiranya Guttala Begumpet Madhapur Hyderabad 500081 India
Telephone : +91 40 33574705
Email ID : info@sanmed.in

Factory: Plot No: 56, Biotech Park-3, Karakapatta, Medak Dist, A.P, India



- ❖ Pour the Strumen™2% & Strumen™ 2.45% in an instrument tray.
- ❖ Activate with Activator provided.
- ❖ To activate 5 Ltr. Add total content of activator bottle.
- ❖ To activate less than 5 Ltr. add proportionately the activator.
- ❖ The solution is now ready for use.
- ❖ Immerse the instruments and scopes in prepared Strumen solution.

Immersion Time

For Total Disinfection : 10- 15 mins (HBV- 30 mins.)

For Total Sterilization : 10 hrs.

RE-USE PERIOD 28 DAYS (LONG LIFE)

Area of application

STRUMEN™ is used for Disinfection and Sterilization of

- ❖ Rigid and flexible endoscopes.
- ❖ Instruments made of metal, porcelain, glass, plastic, stainless steel, ceramics, and rubber.
- ❖ Dimond cutting burs, root canal and hand set instruments etc.

Packing Size

- ❖ 500 ml.
- ❖ 5 Ltr.

www.sanmed.in